

BEE-PATCH - First anti-venom treatment after wasp, bee, hornet, and bumblebee stings

PRODUCT CHARACTERISTICS

- Use:** Bee-Patch soothes the pain, swelling and itching of bee, wasp, hornet, bumblebee stings. Effective on pain and swelling within minutes.
- Composition:** Each patch contains approximately 220 mg of sugar (sucrose). The poison is extracted from the skin through the puncture of the sting by the natural and physical principle of osmosis and transported into the patch.
- Dimensions:** 57 x 58 mm
- Ref article:** PHBP1004017
- EAN / ACL:** 5710391004017
- Shelflife :** 4 years



PRODUCT DESCRIPTION & BENEFITS

- Unique after-bites treatment that works by the natural and physical principle of osmosis.
- Simple and effective solution: efficacy studies with bees and wasps show a reduction of pain within minutes and up to 86% after one hour. After 24 hours, there is a reduction of pain up to 93% and a reduction of swelling up to 99.7%. We also note a decrease of itching.
- Danish brand: in Scandinavian countries, the use of sugar on wasp or bee stings has always been known.
- Patent registered in France (EP/FR number 2056672)
- Medical device class I - CE marking
- Non-allergenic and no side effects (no transcutaneous absorption)
- Recommended for the whole family including children and babies without age restrictions.
- Bee-Patch can easily be slipped into a pocket and can be taken everywhere: hiking, traveling, picnics, outdoor adventures, camping, summer camps...

RECOMMENDATIONS

Remove the sting from the skin (bees) as soon as possible and apply Bee-Patch within the 30 minutes following the sting. Remove the protective paper from the patch and drip a drop of water or spit on the sugar pad in the centre of the patch. Place the patch on the injured skin and press the patch firmly. Let it sit for an hour.

PRECAUTIONS FOR USE

For single-use and external treatment only. Use immediately after opening. If the skin becomes irritated, remove the patch. If you are allergic to bee and wasp stings, always follow your doctor's recommendations. The sugar is not absorbed in the body and can therefore be used by allergy sufferers including diabetics. Used packaging is disposed of in household waste. Store dry and not in direct sunlight.

Designed in Denmark: RFX - Care International A/S - Bøkkkegaardsvej 408 - 3050 Humlebaek - Denmark

Distributed in France: **Katadyn France** | 5 Rue Gallice | 38100 Grenoble | France

Tel: +33 476 964 246 Fax: +33,476 963,981

info@katadyn.fr | www.katadyn.fr | www.pharmavoyage.fr