

### Patch for the treatment of stings from Bees and Wasps – Medicine free

Active substance: Sugar membrane (sucrose)  
Effect: Removes the poison after bee and wasp sting  
Use: Place patch on injured skin and leave for 1 hour  
Safety: If you are allergic to insect sting always follow your Doctors recommendation  
Storage: Store in a dry place 15 – 30 °C  
Shelf-life: 4 years from date of manufacture



### FACTS

- ✔ The patch for instant treatment of stings from bee, wasp, hornet and bumblebee.
- ✔ The patch contains a patented sugar membrane which removes the poison by osmosis.
- ✔ The patch contains ~200 mg sugar (sucrose).
- ✔ The sugar does not cross the skin barrier and can be used by diabetics as well.
- ✔ Both children and adults can use the patch.
- ✔ The patch is developed in Denmark.

### PRODUCT FORM

Patch containing a sugar membrane sealed treatment. Packed in 5 pcs. cardboard box.

### USE

Remove the stinger from the skin if it is still present. Unwrap the patch, add a drop of water in the middle of patch, place the patch on the injured skin. The poison is extracted from the small puncture of the skin by osmosis and transported into the patch. Bee-Patch lowers hereby the pain, swelling and itching of bee, wasp, hornet and bumblebee stings. (Reduction in pain: 5 min 37%, 10 min 58%, 30 min 76%, 1 hour 86%).\*

The patch should remain on the injured skin for 1 hour.

### SAFETY

Single use only. External use only. Use immediately after opening protective wrapper. If skin becomes irritated or infected remove patch and consult a doctor immediately. If you are allergic to insect stings, always follow the instruction from your Doctor. The sugar is not absorbed in the body and can be used by diabetics.

### TESTS

Skin Sensitization test of Bee-Patch: Sanitation & Environment Technology Institute, Soochow University, Report number: SDWH-M20161545-1, July 2020.

Skin irritation test of Bee-Patch: Sanitation & Environment Technology Institute, Soochow University, Report number: SDWH-M20161545-2, July 2020.

\*Post Market Clinical Follow-up: Bee-Patch ApS, October 2020.

Bee-Patch ApS, Nørregårdsvej 14, 4320 Lejre, Denmark.

Tel: +45 2165 9416 [info@bee-patch.com](mailto:info@bee-patch.com) [www.bee-patch.com](http://www.bee-patch.com)